




Protect, Support, Correct




FS3000 FOOT DROP ORTHOTIC (AFO)
THERMOFORMING ADJUSTMENTS GUIDELINE




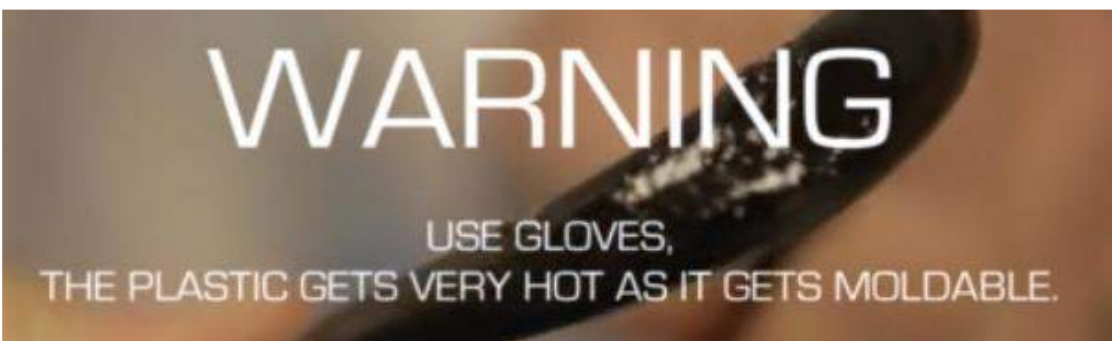


REFER TO THERMOFORMING VIDEO TUTORIAL ON THE WEBSITE

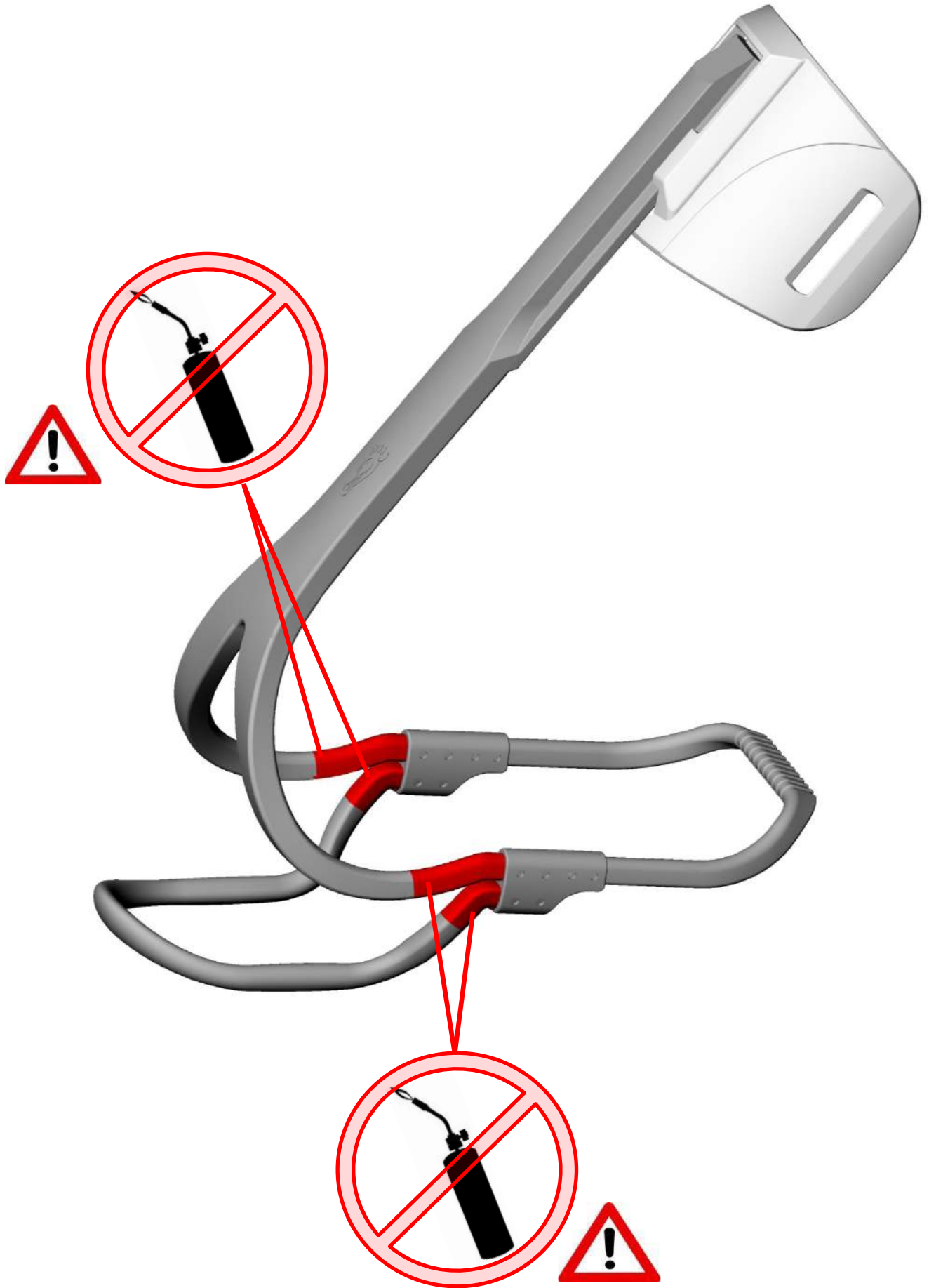


DORSIFLEXION ADJUSTMENT

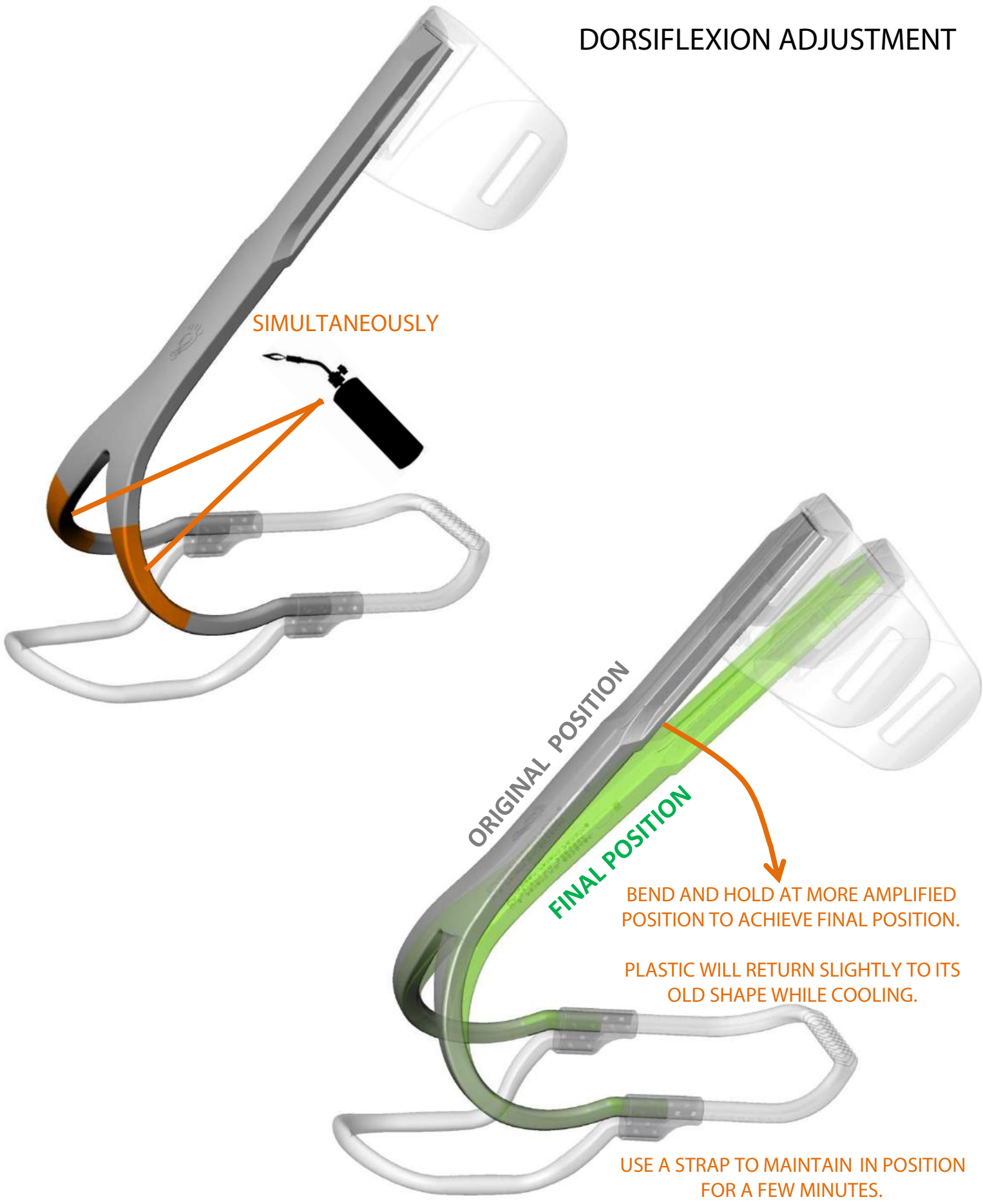

WIDTH ADJUSTMENT WIDER OR NARROWER

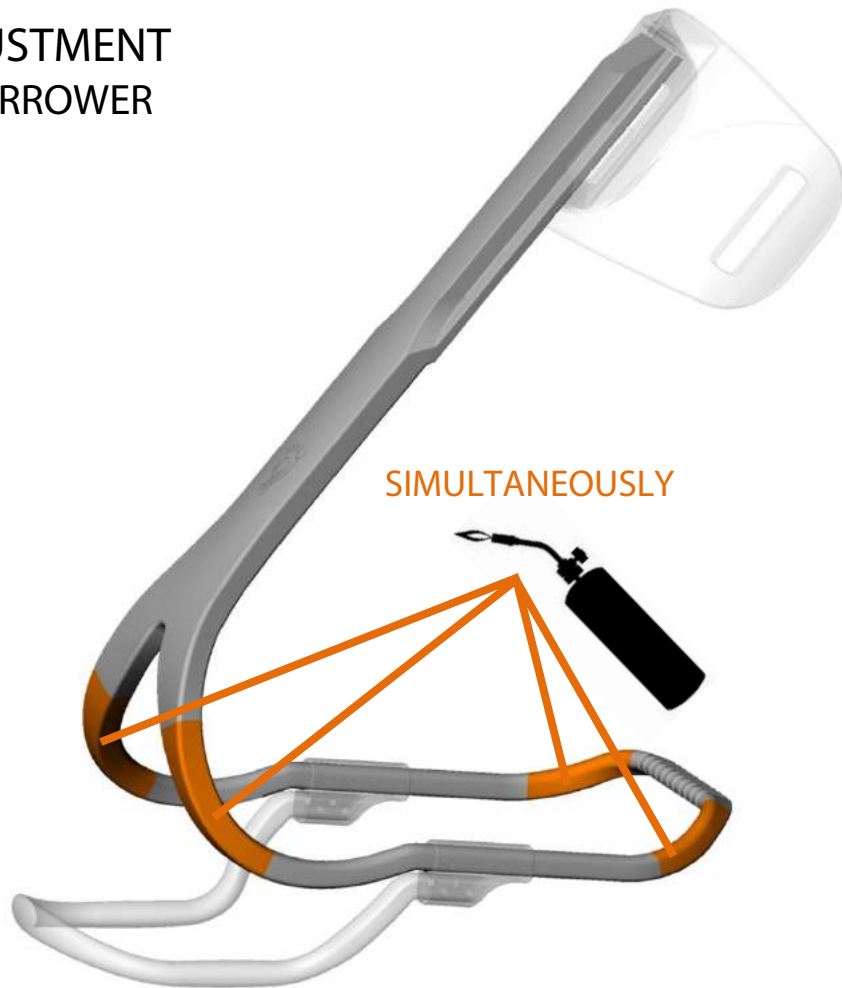

SIMULTANEOUSLY

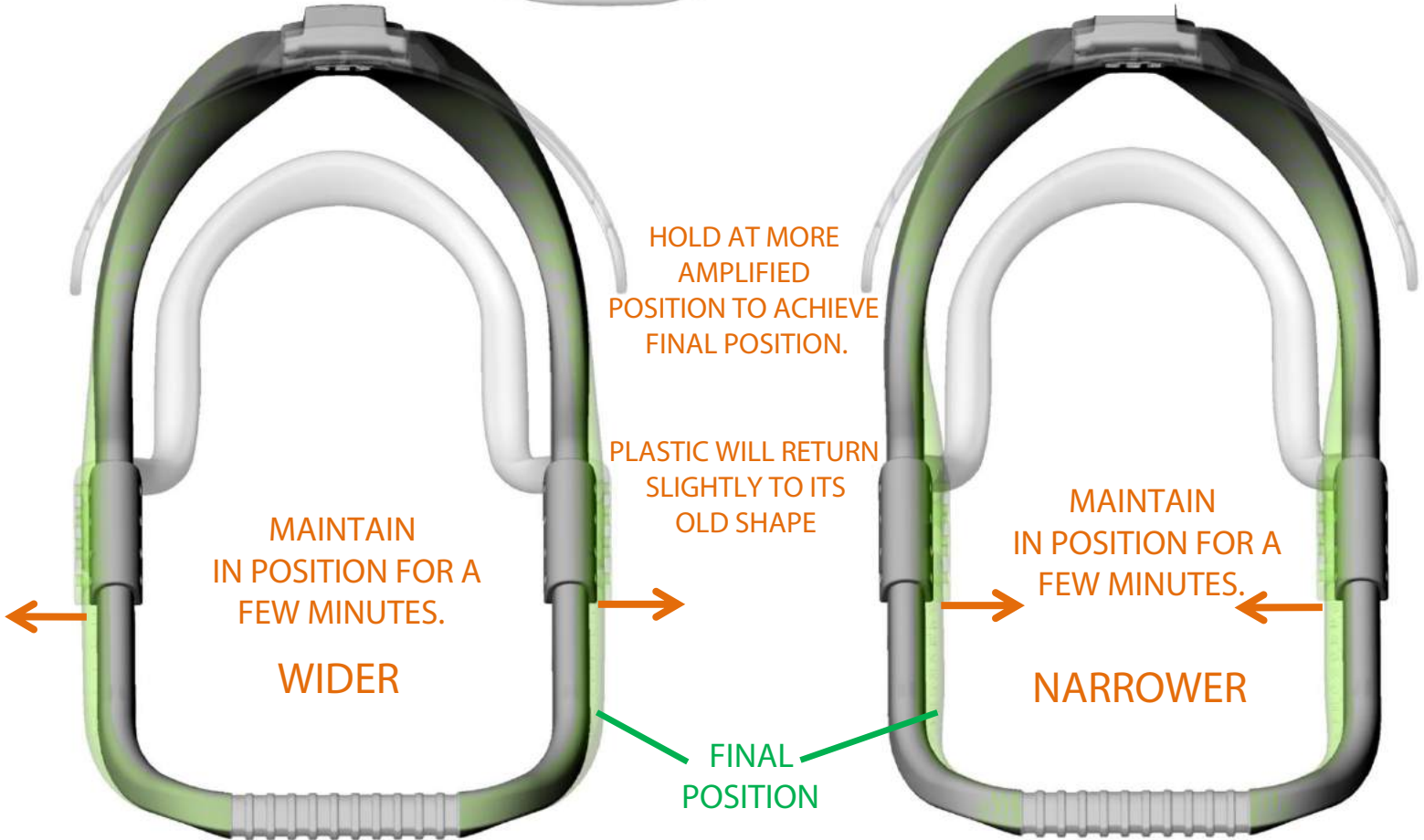

HOLD AT MORE AMPLIFIED POSITION TO ACHIEVE FINAL POSITION.

PLASTIC WILL RETURN SLIGHTLY TO ITS OLD SHAPE

MAINTAIN IN POSITION FOR A FEW MINUTES.



WIDER

MAINTAIN IN POSITION FOR A FEW MINUTES.



NARROWER

FINAL POSITION



SOLE GRIP ADJUSTMENT WIDER OR NARROWER

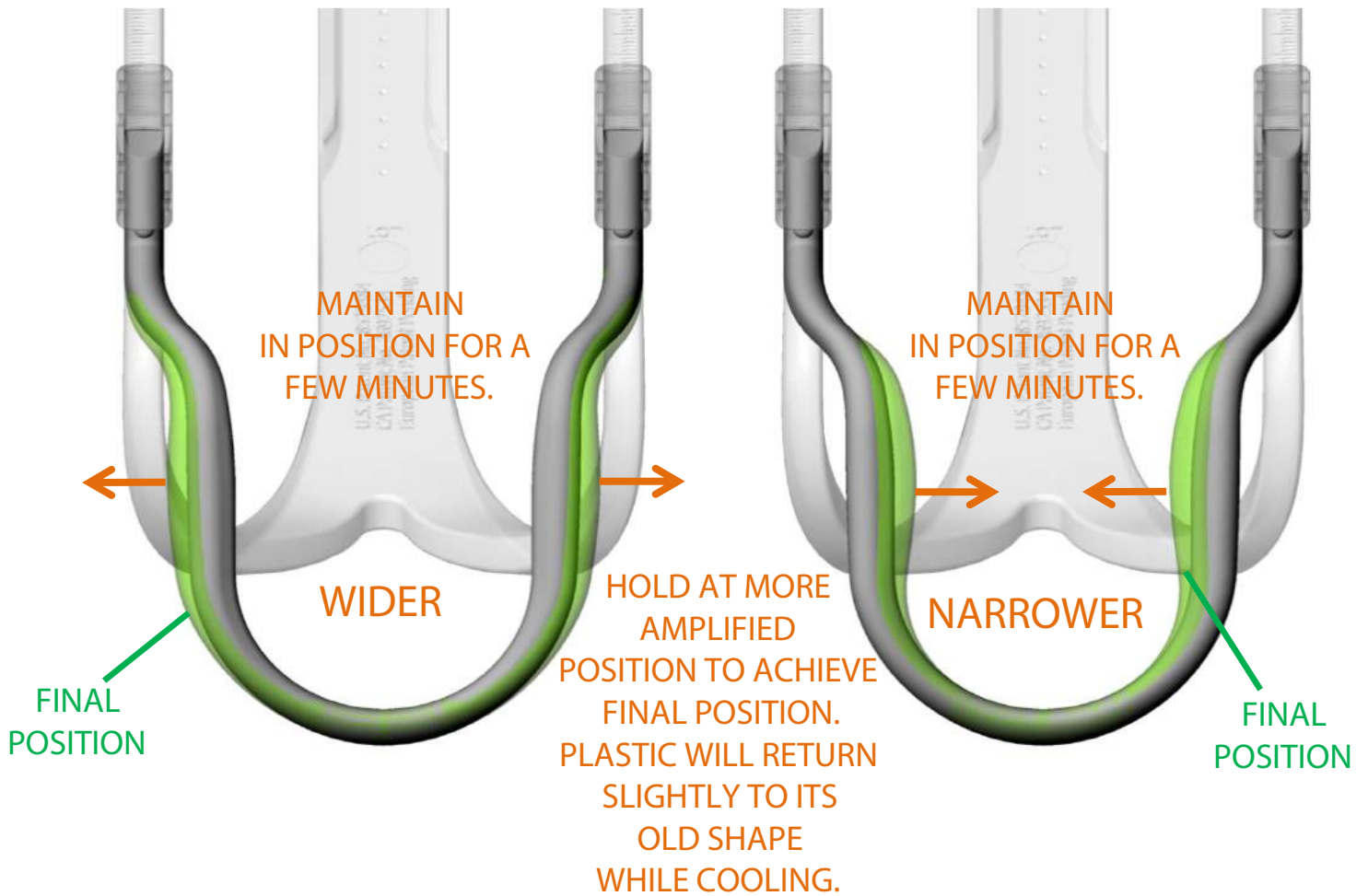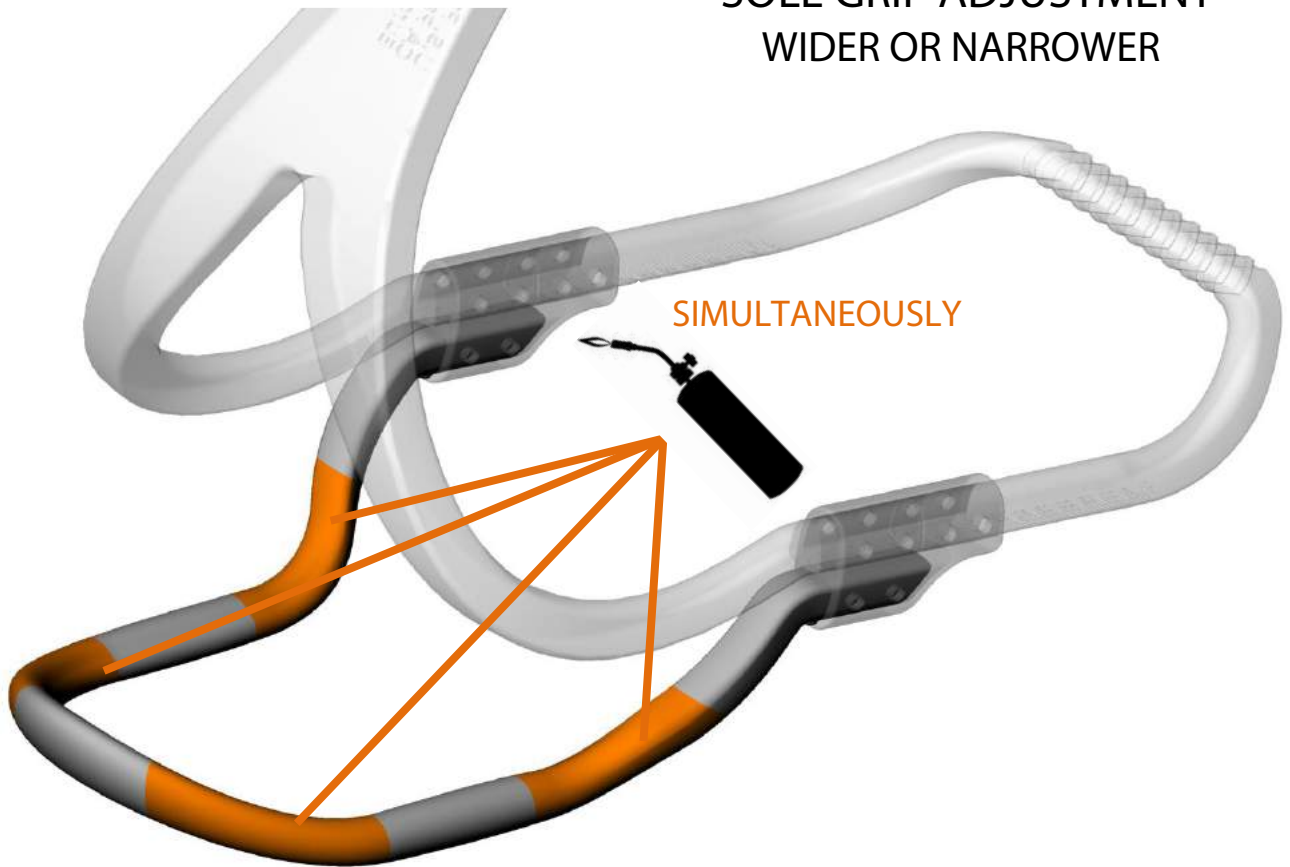

BRACE CENTERING - SOLE GRIP INWARD - OUTWARD

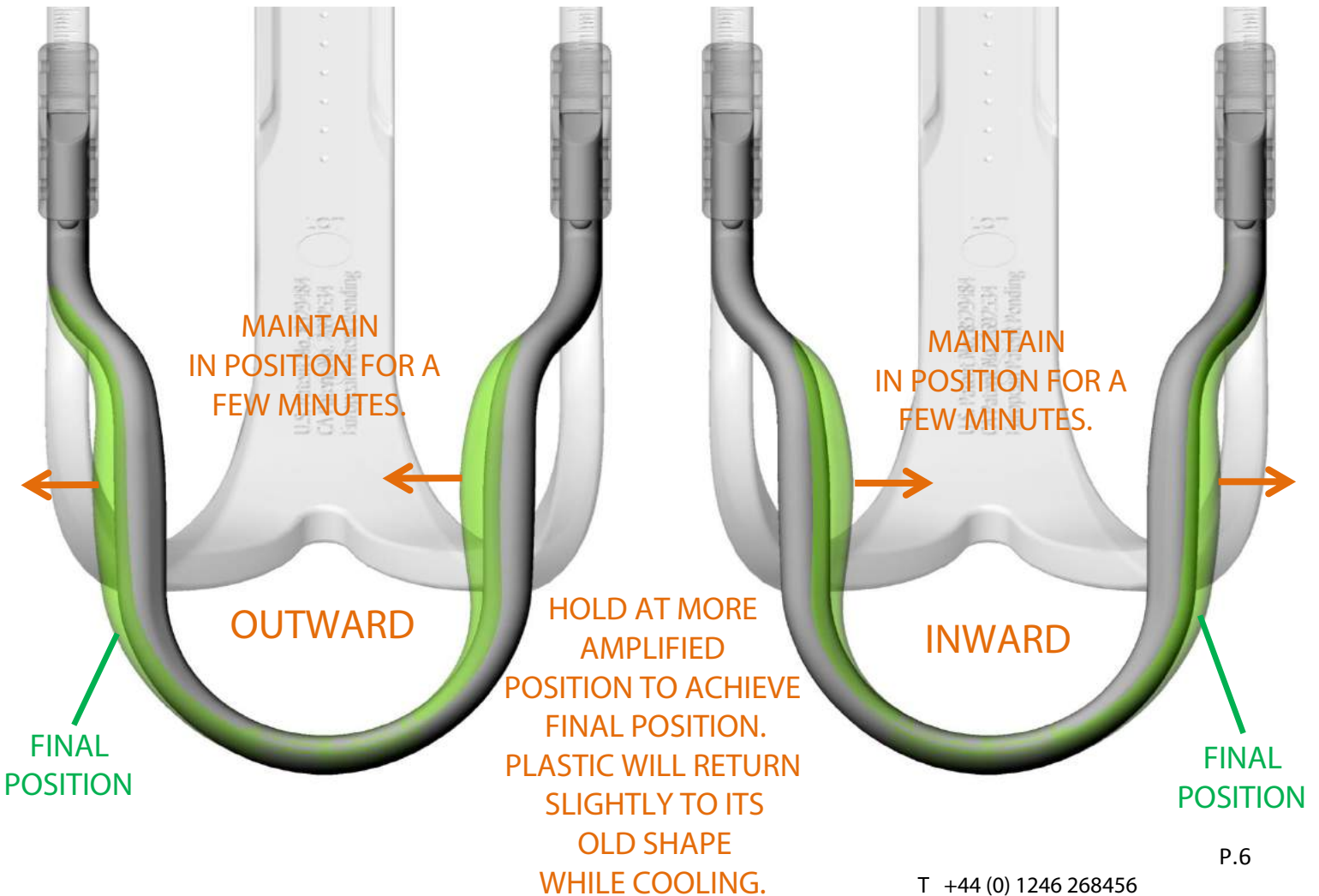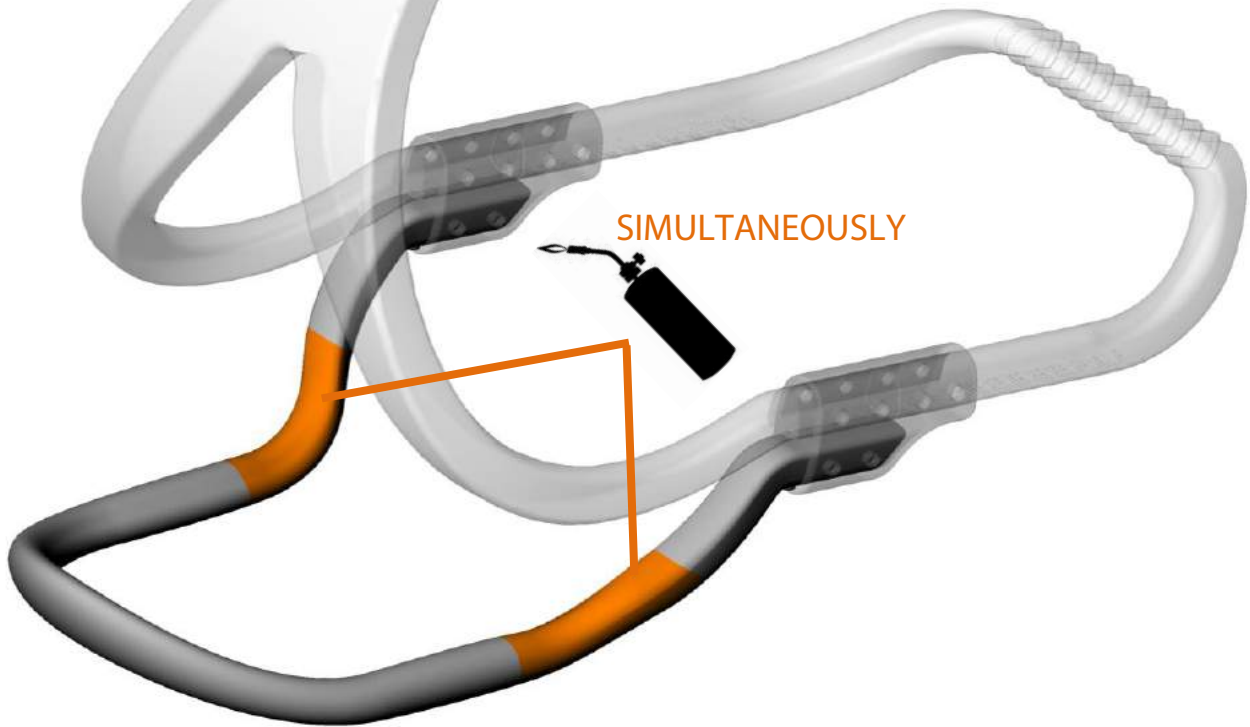

LACECLIP REST ANGLE (PARRALLELISM TOP FOOT FACE)

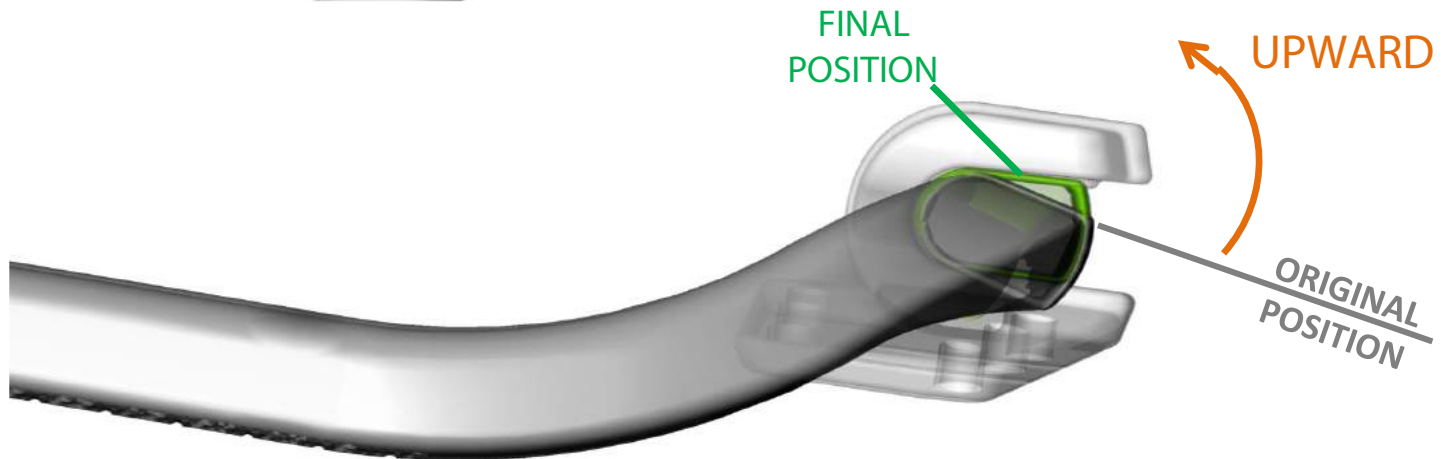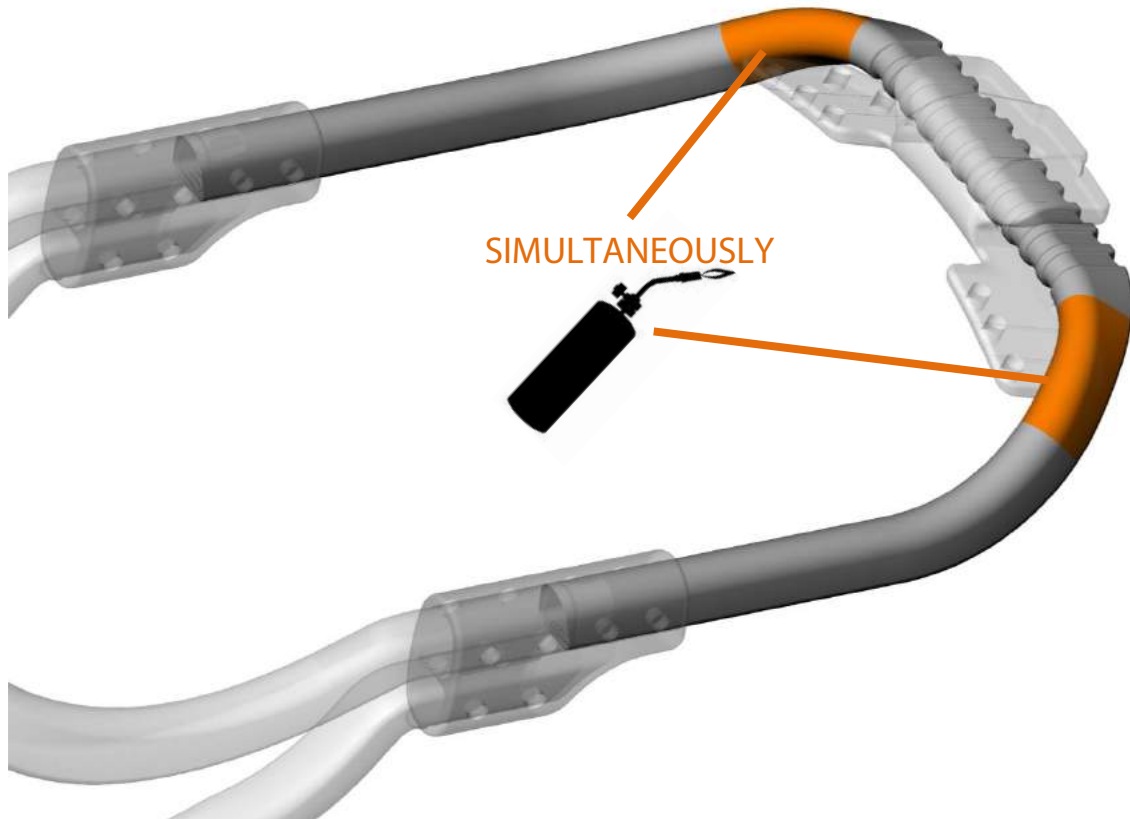

HOLD AT MORE AMPLIFIED POSITION
TO ACHIEVE FINAL POSITION.
PLASTIC WILL RETURN SLIGHTLY TO ITS
OLD SHAPE WHILE COOLING.



CALF SUPPORT ADJUSTMENT

